


durex[®] FIT FINDER

MEASURE UP TO PLEASURE UP

First, print this page at 100% (works in colour or black and white).
Do not scale to fit. To make sure you printed the **FitFinder** at 100%, place a credit or ID card in the box to the right. If it's an exact fit, you're ready to find **your** best fit.

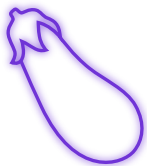

1

GET FULLY ERECT

Cut out the **FitFinder** tool and cut a slit along the dotted line. Then relax, get yourself comfortable and make sure your penis is erect.



2

MEASURE UP

Wrap the narrow end of the **FitFinder** tool around the mid part of your erect shaft and pull the tool through the slit to line the zones up - **X** marks the spot. Refer to the diagram for help.

*Watch out for sharp edges and paper cuts!




3

GET YOUR FIT

Make a note of your fit: **CLOSE**, **REGULAR**, or **WIDE**.

Remember, the output of this tool is just a guide. If the guided size is inconsistent to what you normally buy and the new size doesn't feel comfortable or secure, then you shouldn't use them.

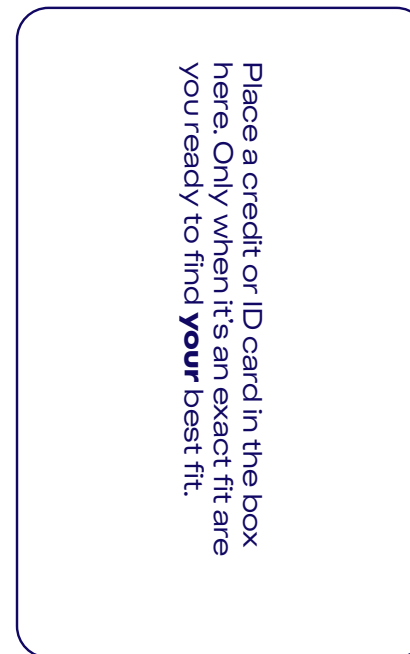

Scan the QR code to shop your size



CLOSE



REGULAR



WIDE

